

Replica™

Patient Care in the Palm of Your Hand



Replica is a mobile application for supplemental remote monitoring and clinician notification that allows clinicians to view continuous monitoring data for multiple patients, as well as view and respond to alarms and alerts, from a smart phone, regardless of location.

- > Displays near real-time data, including high-fidelity waveforms, from connected Masimo and third-party devices, helping clinicians more efficiently manage multiple patients through supplemental remote monitoring
- > Actionable two-way alarm notifications from Masimo and third-party devices are sent to active mobile devices, reaching on-duty and available clinicians, with intelligent escalation that:
 - Allows clinicians to quickly accept or forward alarms, helping improve clinical collaboration
 - Enables automatic escalation to help ensure timely response




Smarter Patient Data Management


With the increasing prevalence of high patient-to-clinician ratios, continuous visibility of the status of multiple patients on a mobile device can help clinicians manage patients more efficiently

At a glance, review the status of all patients on the unit or a smaller subset of assigned patients

Color-coded alarm status allows clinicians to quickly prioritize care, helping improve workflows

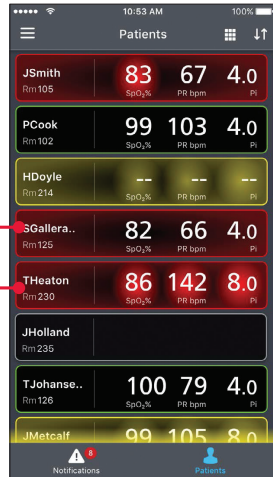

Up to 96 hours of trend data enables clinicians to review changes in a patient's status over time

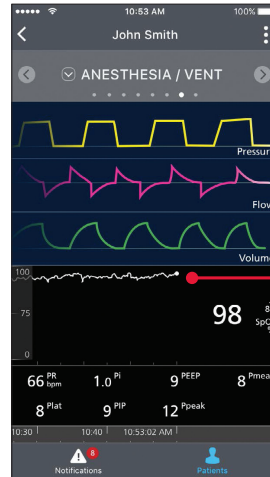



Intelligent Alarm Management

Actionable patient alarms are sent directly to clinicians, with automatic escalation, which can help clinicians respond to critical events more quickly and efficiently

Arrows quickly identify the level of escalation, helping clinicians prioritize response to patients who may need more immediate attention

Clinicians can quickly accept or forward a notification, prompting immediate escalation to the next available clinician, helping coordinate timely response

Detailed timeline of alarms, events, and interventions can help clinicians examine the progression or regression of a patient's status before arriving at the bedside, allowing more time to prepare a clinical plan

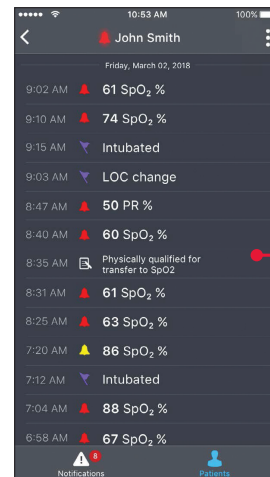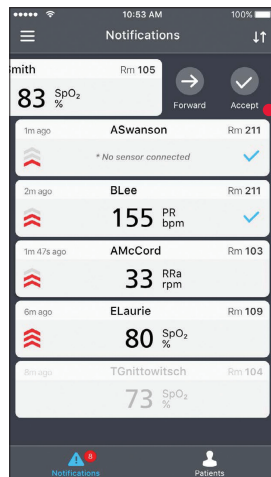

Specifications

SOFTWARE REQUIREMENT

Replica requires Patient SafetyNet™* Series v5.5.4.4 or higher
 Android OS Minimum Version 5.0 (Lollipop)

ORDERING INFORMATION

Replica Software 9863-LIC or 9863-SaaS

* The use of the trademark PATIENT SAFETYNET is under license from University HealthSystem Consortium

Replica is not licensed for sale in Canada.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, and precautions.

Masimo U.S.
 Tel: 1 877 4 Masimo
 info-america@masimo.com

Masimo International
 Tel: +41 32 720 1111
 info-international@masimo.com

